

Quick Start Guide



1 POWER UP

- Lock back wheels before proceeding to Power Up

2 START SOFTWARE

- Press gray **ON/OFF** button until audible beep is heard

3 CALIBRATE

- Verify there is nothing in the BioVision x-ray chamber
- Double click the **VISION** icon in the center of the desktop
- Enter the **PASSWORD**

4 PATIENT DATA

- Calibration will occur automatically once you enter the software

Select:

- Start Procedure
- Patient Data
- Select from Worklist (alternatively you may enter patient data manually)
- Locate patient's name and double click it
- **START**

5 X-RAY IMAGING

- Place specimen(s) in the BioVision x-ray chamber - use magnification tray when possible
- Press the green **X-Ray** button to activate x-rays – image will appear after a few seconds

6 IMAGE MANIPULATION

- Hold down the **left mouse button** and drag the mouse to change brightness and contrast for window/level function
- **Adjust Zoom** to enlarge the image for viewing

7 STORING TO PACS

- Choose **PACS Store** button on left side of screen

8 PRINTING

- Choose **PACS Print** button on left side of screen

9 SHUT DOWN

- Click **EXIT**, Select **SYSTEM SHUTDOWN**, then click **OK**
- An audible beep is heard when the system has completed shutdown